

## > Key properties

- Broad spectrum of microbiological activity
- Fast acting time
- Cleans and disinfects in one
- Formula tested under dermatological control

## > Areas of applications

Aseptonet products are designed to clean and disinfect quickly non invasive medical devices (stethoscopes, blood pressure cuffs, bed frames etc.) in hospitals, medical/dental offices, laboratories, nursing homes, ambulances etc.

## > Active ingredients of the solution

Alcohol, benzalkonium chloride, isopropyl alcohol



## ➤ Microbiological activity

The impregnation solution is :

**Acting time**  
in clean conditions

Microbiological activity	Acting time
<ul style="list-style-type: none"> <li>• <b>Bactericidal</b></li> </ul>	
EN 13697, EN 1276*, EN 14561*	5 min
EN 14561 on MRSA	
<ul style="list-style-type: none"> <li>• <b>Fungicidal</b></li> </ul>	
EN 13697 on Aspergillus niger	5 min
EN 14562 on Aspergillus fumigatus	
<ul style="list-style-type: none"> <li>• <b>Yeasticidal on Candida albicans</b></li> </ul>	
EN 1650*	5 min
EN 13624, EN 13697, EN 14562	
<ul style="list-style-type: none"> <li>• <b>Tuberculocidal on Mycobacterium terrae</b></li> </ul>	
EN 14348, EN 14563	15 min
<ul style="list-style-type: none"> <li>• <b>Virucidal</b></li> </ul>	
EN 14476 on HBV, HCV, Adenovirus, Coronavirus, Norovirus, VRS, Polyomavirus, H1N1	1 min
EN 14476* on HSV	1 min
EN 14476 on Rotavirus	15 min

\* In dirty conditions

## Packaging

Product description	Units per box	Units per pallet
Tub of 100 wet wipes	12	1080
Refill of 100 wet wipes	12	1596
Flow pack of 100 wet wipes	12	600
Foam spray 750 ml	6	450
Can 5 l	1	120

## Direction for use

- **Wipes** : Single-use product. Ready-to-use product. Use Aseptonet wipes on the device to be disinfected. Leave on for a minimum of 5 minutes and let dry. Do not rinse.
- **Spray** : Ready-to-use product. Spray Aseptonet on the device to be disinfected and spread evenly. Leave on for a minimum of 5 minutes and let dry. Do not rinse.
- **Can** : Ready-to-use product. Apply Aseptonet on the device to be disinfected and spread evenly. Leave on for a minimum of 5 minutes and let dry. Do not rinse.

## Cautions

Avoid contact with open wounds, irritated skin or mucous membranes. Keep out of reach of children.